ASSEMBLY NOTICE

Cod.	CPS202
Product:	DROP EYE WASH STATION
CE:	MDR 745/2017 of 05/04/2017

English Instructions

Mounting and Usage Instructions

Required Tools:

- Drill with a bit suitable for your wall type
- Screwdriver
- Spirit level
- Pencil
- Measuring tape
- Screws and wall plugs (included in the kit)

Step 1: Prepare the Eyewash Station

1. Remove the DROP station from its packaging.
2. Ensure all components are present:
 - Mirror
 - Two bottles (500 ml sterile saline solution and 250 ml pH neutral solution)
 - Screws and wall plugs

Step 2: Choose the Location

1. Select an easily accessible and visible location, preferably near high-risk areas.
2. Use the spirit level to ensure the station will be mounted horizontally.

Step 3: Marking and Drilling

1. Hold the station against the wall at the desired height.
2. Use the pencil to mark the drilling points through the pre-drilled holes.
3. Remove the station and drill holes at the marked points.
4. Insert the provided wall plugs into the holes.

Step 4: Mounting the Station

1. Place the station back against the wall, aligning it with the drilled holes.
2. Use the screwdriver to secure the station to the wall with the included screws.
3. Ensure the station is firmly fixed and stable.

Step 5: Usage Instructions

1. **In Case of Eye Emergency:**

- Gently pull down the cover to open the station.
- Remove the required bottle:
 - **500 ml sterile saline solution** for general eye rinsing.
 - **250 ml pH neutral solution** to neutralize acids or alkalis.
- Twist the cap to break the seal.
- Use the ergonomic eye cup to direct the fluid into the affected eye.

2. **Post-Use:**

- Use the integrated mirror to assess any damage or remaining contaminants.
 - Seek medical attention if necessary.
-

Maintenance and Refilling

• **Regular Checks:**

- Inspect the station periodically to ensure it's in good condition.
- Check the expiration dates of the solutions.

• **Refilling:**

- Replace used or expired bottles with:
 - **ACQ414:** 500 ml sterile saline solution.
 - **ACQ406:** 250 ml pH neutral solution.
-

Safety Notes

- Ensure all personnel are trained on the location and proper use of the eyewash station.
 - Do not block access to the station with obstacles.
-

Instructions de Montage

Cod.	CPS202
Product:	DROP EYE WASH STATION
CE:	MDR 745/2017 du 05/04/2017

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